



VitaVitro

VitaVitro® Culture Dish User Manual



CAUTION

Federal (U.S.) law restricts this device to sale by or on the order of a physician.

INDICATIONS FOR USE

Intended for the preparation, storage, manipulation or transfer of human gametes or embryos for in vitro fertilization or other assisted reproductive procedures.

TYPE

35mm Culture Dish
60mm Culture Dish
90mm Culture Dish
100mm Culture Dish
Central Well Culture Dish
4-Well Culture Dish
Well of the Well Dish
Humdish

WARNING

1) This device to sale by or on the order of a physician (or properly licensed practitioner).

- 2) The user should read and understand the Instructions for Use, Warning, and be trained in the correct procedure before using.
- 3) Do not use the product if the product packaging appears damaged or broken.
- 4) For single use only.
- 5) Do not use after expiry date.
- 6) To avoid problems with contamination, practice aseptic techniques.
- 7) Discard the dish according to standard clinical practice for medical hazardous waste when the procedure is finished.

STORAGE AND SHELF LIFE

Store at room temperature. The shelf life is 3 year from time of manufacture.

COMPOSITION

The VitaVitro® Culture Dish is constructed of polystyrene and is non-pyrogenic. It has passed USP class VI testing and is sterilized by gamma irradiation.

QUALITY CONTROL TESTING

Sterility tested Passed
LAL Endotoxin < 20 EU/device
1-cell Mouse Embryo Assay
(% blastocysts at 96 h of culture) ≥ 80%

INSTRUCTION FOR USE









1) Open the packing

The packing must be opened in a sterile environment at ambient temperature and humidity. Keep the lid closed before use.

2) Follow the procedures prescribed by the assisted reproduction laboratory to use this product.

DESCRIPTION OF ISO SYMBOLS

The symbol glossary is in line with the SDO-developed standard ANSI/AAMI/ISO 15223-1: Medical devices - Symbols to be used with medical device labels, labeling and information to be supplied-Part 1: General requirements.

Reference number of symbol	Symbols	Title of symbol
5.1.1		Manufacturer
5.1.3		Date of manufacture
5.1.4		Use-by date
5.1.5		Batch code
5.2.4		Sterilized using irradiation
5.2.8		Do not use if package is damaged
5.3.2		Keep away from sunlight
5.4.2		Do not re-use



VitaVitra

 **Shenzhen VitaVitra Biotech Co., Ltd.**

R601, Building B, and 301, Area 02,
Building A, Hai Ke Xing Tech Park
Baoshan Road No.16, Pingshan
District, Shenzhen, Guangdong,
518118, China

Phone: +86 755 84511813

e-mail: tech@vitavitra.com

www.vitavitra.com